



hydroven[®]

Garment Measuring Guide

HUNTLEIGH

A MEMBER OF THE ARJO FAMILY

Garment Measuring Guide


The following guide is intended as an aid to help the user select the correct size garment.

The treatment recommended by a HC professional will determine the garment type e.g. hydroven® FPR for sequential therapy, hydroven® 1 for uniform compression, hydroven® 3 for graduated treatment and or hydroven® 12 LymphAssist for specialist multi chamber treatment.

When determining the correct size garment:

- Remove any bandaging, stockings or bulky dressings
- Measure the points as shown on the diagram below
- DO NOT pull the measuring tape tight
- Make sure the garment extends beyond the area under treatment

Refer to the garment list at the end of this document.

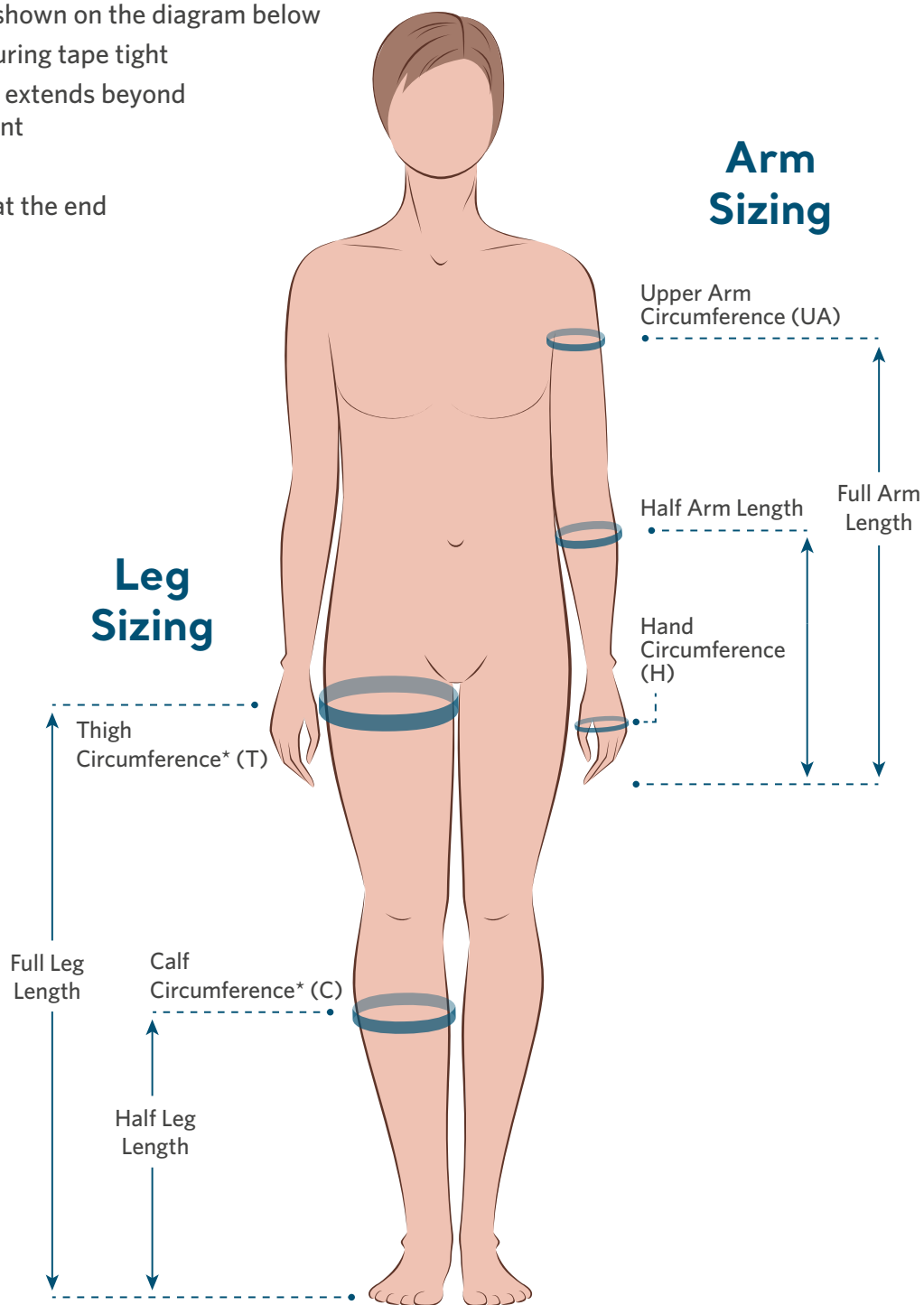

*Measurements should be made at the widest point on the calf and thigh.

Garment Measuring Guide

Hydroven FPR Leg

Three Chamber Leg Garments

• Sequential Therapy



Order Code	Type	Length (L)	Calf Circumference (W)
5163L50	Half Leg	20"	24"

Order Code	Type	Length (L)	Thigh Circumference (W)
5163L66	Full Leg	26"	25"
5163L71	Full Leg	28"	26"
5163L76	Full Leg	30"	28"
5163L84	Full Leg	33"	28"
5163L92	Full Leg	36"	28"

• Sequential Therapy



Hydroven FPR Arm

Three Chamber Arm Garments

Order Code	Type	Length (L)	Circumference Hand (H)	Circumference Shoulder (S)
5163A68	Full Arm	26"	17"	24"
5163A78	Full Arm	30"	17"	24"

Garment Measuring Guide

Hydroven 1 Leg

Single Chamber Leg Garments

▪ Uniform Therapy




Order Code	Type	Length (L)	Calf Circumference (W)
5101L50	Half Leg	20"	24"

Order Code	Type	Length (L)	Thigh Circumference (W)
5101L66	Full Leg	26"	25"
5101L71	Full Leg	28"	26"
5101L76	Full Leg	30"	28"
5101L84	Full Leg	33"	28"
5101L92	Full Leg	36"	28"

Hydroven 1 Arm

Single Chamber Arm Garments

▪ Uniform Therapy

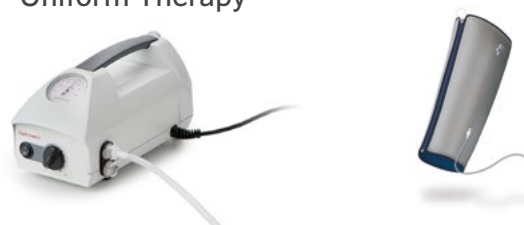

Order Code	Type	Length (L)	Circumference Hand (H)	Circumference Below Elbow (S)
5101A51	Half Arm	20"	17"	22"

Order Code	Type	Length (L)	Circumference Hand (H)	Circumference Shoulder (S)
5101A68	Full Arm	26"	17"	24"
5101A78	Full Arm	30"	17"	24"

Garment Measuring Guide

Hydroven 3 Leg

Three Chamber Leg Garments

- Graduated Therapy



Order Code	Type	Length (L)	Calf Circumference (W)
5103L50	Half Leg	20"	24"

Order Code	Type	Length (L)	Thigh Circumference (W)
5103L66	Full Leg	26"	25"
5103L71	Full Leg	28"	26"
5103L76	Full Leg	30"	28"
5103L84	Full Leg	33"	28"
5103L92	Full Leg	36"	28"

Hydroven 3 Arm

Three Chamber Arm Garments

- Graduated Therapy



Order Code	Type	Length (L)	Circumference Hand (H)	Circumference Shoulder (S)
5103A68	Full Arm	26"	17"	24"
5103A78	Full Arm	30"	17"	24"

Hydroven Expanders

Compatible with Hydroven FPR, Hydroven 1 and Hydroven 3 garments



Order Code	Type	Length (L)	Added Circumference Wide End	Added Circumference Narrow End
Leg Inserts				
510LI50	Half Leg Insert	20"	7½"	5½"
510LI66	Full Leg Insert	26"	7½"	5½"
510LI71	Full Leg Insert	28"	7½"	5½"
510LI76	Full Leg Insert	30"	7½"	5½"
510LI84	Full Leg Insert	33"	7½"	5½"
510LI92	Full Leg Insert	36"	7½"	5½"
Arm Inserts				
510AI68	Full Arm Insert	26"	6½"	5"
510AI78	Full Arm Insert	30"	6½"	5"

Garment Measuring Guide

Lymph Assist (Hydroven 12) Leg

Twelve Chamber Leg Garments



Order Code	Type	Length (L)	Thigh Circumference
316L76S	Standard Leg	30"	28"
316L84S	Standard Leg	33"	28"
316L76W	Wide Leg	30"	31"
316L84W	Wide Leg	33"	31"

Lymph Assist (Hydroven 12) Leg Expanders

Twelve Chamber Leg Inserts



Order Code	Type	Length (L)	Additional Width at Thigh
316LI76	Leg Insert	30"	7½"
316LI84	Leg Insert	33"	7½"

Lymph Assist (Hydroven 12) Arm

Twelve Chamber Arm Garments



Order Code	Type	Length (L)	Circumference Shoulder (H)
316A68	Full Arm	26"	24"
316A78	Full Arm	30"	24"

Lymph Assist (Hydroven 12) Arm Expanders

Twelve Chamber Arm Inserts



Order Code	Type	Length (L)	Additional Width at Shoulder
316AI68	Full Arm	26"	6½"
316AI78	Full Arm	30"	6½"

Garment Measuring Guide

Notes:

This image shows a blank sheet of white paper with horizontal ruling lines. The lines are evenly spaced and run across the width of the page. There are no margins, text, or other markings on the paper.

As a proud member of the Arjo family, we have been committed to supporting healthcare professionals in improving outcomes and enhancing patient wellbeing since 1979. We do this through our proven solutions for Vascular Assessment & Treatment and Fetal & Patient Monitoring. With innovation and customer satisfaction as our guiding principles, we strive for clinical excellence and improved performance, for life.

Distributed in the USA by:

Arjo Inc..

2349 West Lake Street, Suite 250 Addison, Illinois 60101

T: +1 (800) 323 1245 F: +1 888 594 2756

www.huntleigh-healthcare.us

Registered No: 942245 England & Wales. Registered Office:

ArjoHuntleigh House, Houghton Hall Business Park, Houghton Regis, Bedfordshire, LU5 5XF

©Huntleigh Healthcare Limited 2015-2020

A Member of the Arjo Family

As our policy is one of continuous improvement, we reserve the right to modify designs without prior notice.

AW:1001047-4

HUNTLEIGH

LIT 779416/LAT-3